D E S
I G N
E Y E
W E A
R G R
O U P

DA

EN

FR

IT

DE

NL

NO

SV

UK

PL

EL

CS

HR

PT

SK

SL

TR

RU

FI

HU

KO

ZH

JA

BG

AR

Etikettering af brillestel

Brugsområde

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Optical frame labelling

Field of use

Spectacle frames are class I medical devices (Rule 1 Chapter III of Annex VIII of (EU) Regulation 2017/745), which are principally intended for use in connection with ophthalmic prescription lenses for correcting visual problems. Optical frames comply with (EU) Regulation 2017/745

Warning

Adverse contact reactions to materials and treatments used for the frames in particularly sensitive or allergic subjects cannot be ruled out. The frame comes with clear lenses that are not intended for use but are designed only to prevent frame warping and allow for a more realistic try-on.

Storage and Cleaning

Store the optical frames in their case at a temperature between -10°C and +35°C. Keep away from direct sunlight and avoid exposure to high temperatures, such as heat inside a car without air conditioning (e.g. on the dashboard), as high temperatures may alter the features of the glasses. In case of contact with agents such as salt water (seawater), swimming pool water, chemical products such as make-up, hairspray, sun lotions, and insect repellents, we recommend that you immediately remove such agents using a soft damp cloth. In case of wear and tear (e.g. scratched, opaque), we recommend the use of original replacement spare parts. Use a damp cloth and mild soap to clean the glasses, then dry them with a soft, clean cloth. Do not use solvents (e.g. alcohol, acetone) or aggressive detergents which may alter the features of the glasses.

Marking

The model identification mark is on the inside of the product. The date of manufacture is indicated on the individual packaging.

Reporting/Contact

In case of complaints or anomalies, should you not be able to contact the store where you purchased the product, please forward your request to:

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Easy Medical Device GmbH